

# Dental Handpiece Lubrication Chart

## High-speed Handpieces

Between Patients



Scrub and rinse handpiece with mild soap and water before lubricating. (Do not wipe down with cleaning/ disinfecting wipes or other harsh chemicals.)

### CLEAN AND LUBRICATE

Remove bur and select the appropriate nozzle. (Selecting the proper nozzle will ensure a tight seal to properly lubricate throughout the handpiece). Insert nozzle into swivel back-end or back of 4/5-hole handpiece.

Spray for one second.

### RUN THE HANDPIECE

Run handpiece on a flush station for 20 seconds. Handpieces requiring a bur tool must be run with the bur installed. If flush station is not installed, run handpiece chairside.

### WIPE AND STERILIZE


Remove bur. Wipe handpiece with paper towel and sterilize.

## Slow-speed Handpieces

Between Patients/  
Weekly Maintenance



Place 1-3 drops from the dropper bottle at all points as indicated on the illustration.

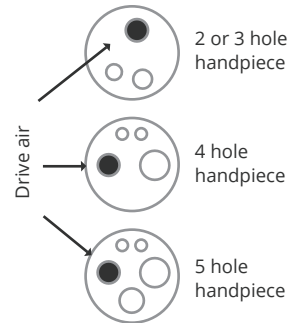

When applying the lubricant into a 4/5-hole handpiece with dropper bottle, use the drive air tube only. Which is smaller than the exhaust tube.

Run handpiece for 20 seconds. Handpieces requiring a bur tool must be run with the bur installed. If flush station is not installed, run handpiece chairside.

Remove bur. Wipe handpiece with paper towel and sterilize.

## Electric Endo & Implant Handpieces

Between Patients



Remove handpiece from motor and bur from handpiece. Never lubricate an electric motor.

Fit spray can with appropriate adapter nozzle for handpiece. Slide lubrication adapter into back of handpiece.

Spray for 2 seconds, holding handpiece and nozzle together and pointing front of head away from your face.

Install bur and run handpiece for 20 seconds.

Remove bur. Wipe handpiece with paper towel and sterilize.

## Attachments

Between Patients/  
Weekly Maintenance



Use dropper to lubricate all moving parts and run briefly.

Remove bur. Wipe handpiece with paper towel and sterilize.

